



## Sephure Medication Sizing Chart

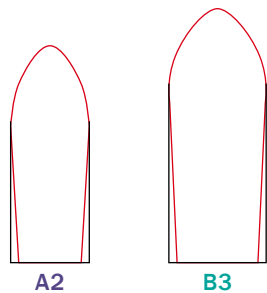


### To Determine Correct Applicator Size:

1. Print this page without scaling on a letter sized sheet  
(use quarter illustration to determine if print out has been scaled)
2. align left most edge of suppository to left edge of chart to the left
3. align bottom of suppository to bottom edge of chart to the left
4. the widest width of the suppository determines which applicator is correct  
(height of suppository may vary from illustrations below depending on the shape)

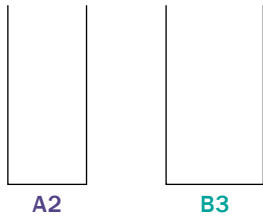

Correct Sizing

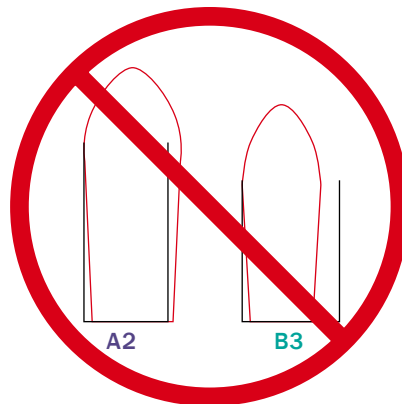

Incorrect Sizing

### Medication List:

Sephure **A2** will fit the following medications:  
(medications of 2.8 to 3.0 total gram weight)

Canasa

Pentasa

Mesalamine

Sephure **B3** will fit the following medications:  
(medications of 2.3 to 2.5 total gram weight)

Laxatives: Dulcolax, Fleet, Magic Bullet  
and generic equivalents

Preparation H and Tucks Internal  
Soothers and generic equivalents

Anusol - HC

Hydrocortisone

If your medication is not listed, give us a  
call at 1-844-SEPHURE and we can help  
determine which size to order